## en OPERATING YOUR SMARTLINKO2 APP

### 1. Download SmartLinkO2 App onto your smartphone

Apple App Store

Google Play App Store

SmartLinkO2 App

### 2. Turn on Bluetooth® in your Smartphone Settings

Apple Settings App

Android Settings App

Bluetooth


### 3. Plug in Your iGo2® POC


### 4. Open SmartLinkO2 App

- Register Your Account.
- Enter your information including desired password and click Register or Sign In to your existing account.

New User or Existing Account Screen

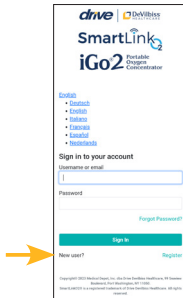
Register Screen
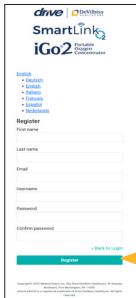
### 5. Pairing

- Select Begin Pairing and follow the pairing wizard steps.
  - If applicable, select "While using the App" for location permissions.
- At Pairing Device screen, move your smartphone toward iGo2 POC screen.
- On Smartphone, confirm iGo2 Serial Number is correct and press Pair. Pairing Successful screen will briefly display.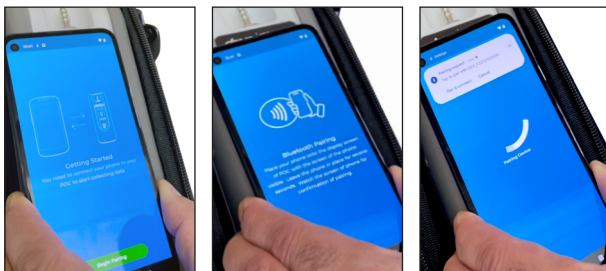


### 6. Viewing and Updating Data

- iGo2 POC information including battery status, power on/off, SmartDose® on/off, prescription setting, oxygen concentration, and breath rate is displayed on Device Status Screen.

**NOTE**-Oxygen concentration indicator displays yellow during warm up and green when start up cycle is complete.


- iGo2 POC information is updated when you open the SmartLinkO2 app. To update after the app is opened, swipe down on the smartphone screen and release. The iGo2 will send updated information to your app.

Device Status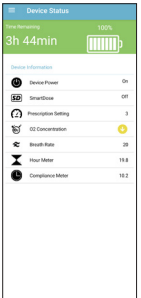

### 7. FAQs

- Select Menu icon on Device Status screen, choose Settings then FAQs for a list of questions and answers.

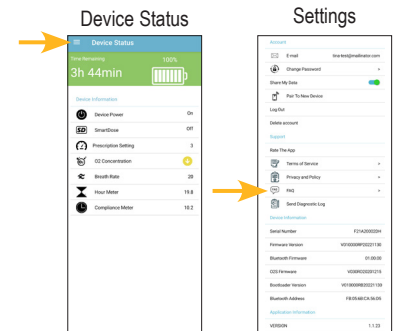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