


Drive Med-Aire Reference Guide

Item # 14027P

For use with 14027 Med-Aire System

for the road ahead™
drive

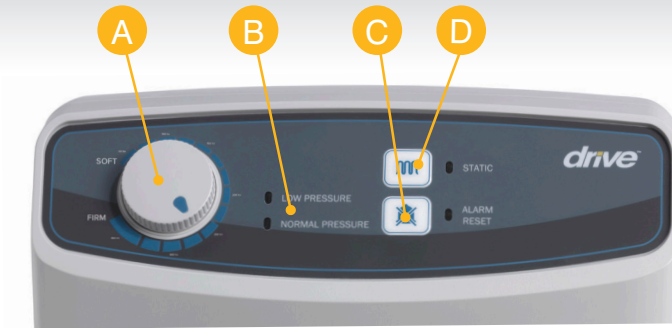

Power Switch

Power switch is located on the side of the control unit. Use the power switch to turn the system off and on.

A. Patient Weight Setting

Allows the caregiver to set the mattress to the patient's weight and desired level of firmness. Use the dial to increase/decrease level of firmness.

B. Normal/Low Pressure

Alerts the caregiver that either the settings are too low for the patient or there is a leak in the mattress.

C. Alarm Mute

Silences the audible portion of the low pressure information signal.

D. Therapy Mode

Press the static pressure button to discontinue alternating therapy.

Set Up

To Safely Transfer the Patient onto the mattress:

1. Adjust dial to max firm to fully inflate the mattress to allow safe patient transfer onto the mattress.

Once the Patient is in the Bed:

2. Alternating Pressure mode is the automatic primary therapy offered for the patient once in the bed.
3. Patient Weight Settings are intended to be used as a guide. Initial setting should be chosen based on the patient weight, comfort, and tolerance, but can be adjusted anytime based on patient comfort level.
4. The alternating pressure cycle time is 10 minutes and can be discontinued at any time by selecting the static button.

For Bed Side Treatment:

5. Static Mode may be used to temporarily suspend alternating pressure mode and to treat the patient bedside: ex. dressing change, bed pan, etc.

To Perform CPR:

6. Pull the CPR pull cord on the mattress for rapid deflation and follow facility protocol

If you have additional questions about your Drive device, please contact an authorized Drive Dealer or Technical Services for support.