


# PRESSURE SUPPORT SURFACE SELECTION ALGORITHM



Support Surfaces are an integral component of pressure injury and treatment but there is insufficient evidence to guide clinical decision making, as per the National Pressure Injury Advisory Panel (NPIAP).

Drive Gravity Therapeutic Foam Mattress (consider patient weight, weight distribution)  
Gravity mattress available in 76" , 80" and 84" lengths and 36" and 42" widths

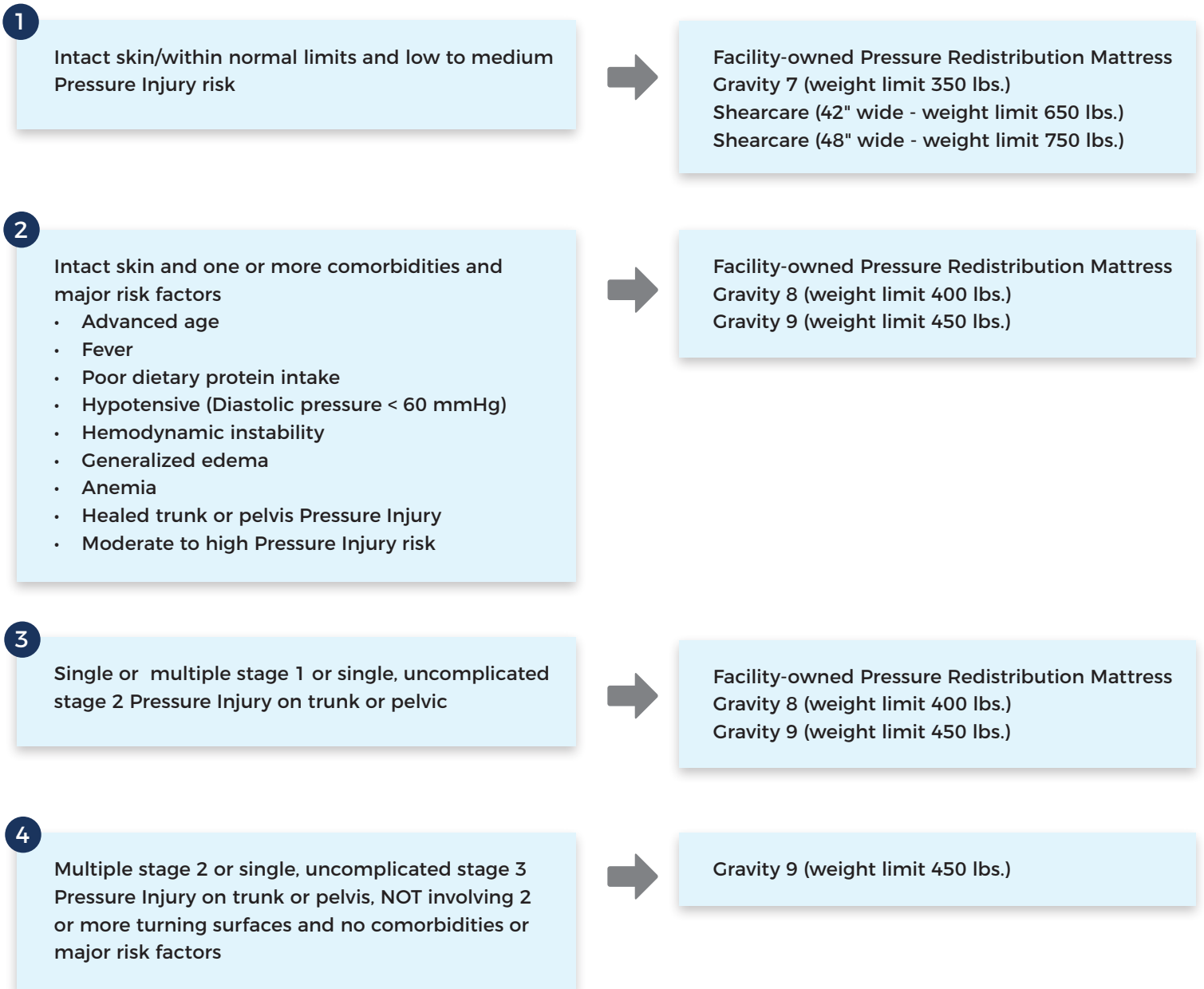

## Powered Low Air Loss/Alternating Pressure Mattress

1

Non intact skin/not within normal limits (one or more)

- Inflammation
- MASD
- Discoloration
- Induration
- Bogginess
- Broken skin
  - Partial thickness
  - Full thickness
- Healed Pressure Injury < 1 year



Drive Med-Aire Plus

14029, 14029-84, 14029DP (weight limit 450 lbs.)  
14030 (42" wide - weight limit 600 lbs.)  
14048 (48" wide - weight limit 750 lbs.)  
14054 (54" wide - weight limit 1,000 lbs.)  
14060 (60" wide - weight limit 1,000 lbs.)

2

At very high risk for Pressure Injury and one or more comorbidities and major risk factors

- Advanced age
- Fever
- Poor dietary protein intake
- Hypotensive (Diastolic pressure < 60 mmHg)
- Hemodynamic instability
- Generalized edema
- Anemia
- Healed trunk or pelvis Pressure Injury



Drive Med-Aire Plus

14029, 14029-84, 14029DP (weight limit 450 lbs.)  
14030 (42" wide - weight limit 600 lbs.)  
14048 (48" wide - weight limit 750 lbs.)  
14054 (54" wide - weight limit 1,000 lbs.)  
14060 (60" wide - weight limit 1,000 lbs.)

3

Multiple Stage 2, stage 3 or 4 or unstageable Pressure Injuries on trunk or pelvic NOT involving 2 or more turning surfaces



Drive Med-Aire Plus

14029, 14029-84, 14029DP (weight limit 450 lbs.)  
14030 (42" wide - weight limit 600 lbs.)  
14048 (48" wide - weight limit 750 lbs.)  
14054 (54" wide - weight limit 1,000 lbs.)  
14060 (60" wide - weight limit 1,000 lbs.)

## Powered Blower Based True Low Air Loss/Alternating Pressure with Pulsation

1

Multiple trunk or pelvic stage 3, stage 4 or unstageable Pressure Injuries involving 2 or more turning surfaces.



Drive Harmony

14200 (36" wide - weight limit 400 lbs.)  
14200-42 (42" wide - weight limit 700 lbs.)  
14200-48 (48" wide - weight limit 900 lbs.)

2

Post-operative flap or graft surgery on trunk or pelvis



Drive Harmony

14200 (36" wide - weight limit 400 lbs.)  
14200-42 (42" wide - weight limit 700 lbs.)  
14200-48 (48" wide - weight limit 900 lbs.)